

# ASSEMBLY UPDATE

## SleepSafer® Bed Extension Latch Installation

*Latching mechanism for the top safety rail of a SleepSafer® Bed provides additional security when properly attached to the SleepSafer® Bed Extension.*



SleepSafe Beds, LLC  
3629 Reed Creek Drive  
Bassett, VA 24055


Toll Free: 866.852.2337  
Fax: 276.627.0234

SleepSafeBed.com  
sleepsafebed@sleepsafebed.com

1



2

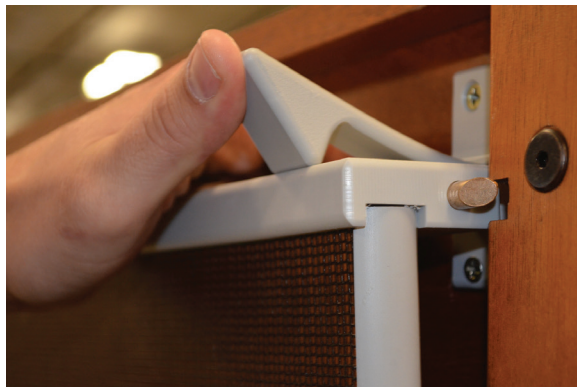

3



### INSTALL LATCH.

On the right side of the SleepSafer® Bed Extension, locate and align the latch holes to the threaded inserts located in the pre-drilled holes. Then, using a phillips head screwdriver, mount latch with provided screws to the SleepSafer® Extension, as pictured.

### PUSH UP LATCH FOR SAFETY RAIL TO SECURE IT TO THE EXTENSION.

With your thumb, push latch up to allow the safety rail to securely fasten to the SleepSafer® Extension as pictured.

### SHOWING LATCH SECURELY HOLDING SAFETY RAIL IN PLACE.

To disengage the safety rail from the SleepSafer® Extension, simply push up the latch to allow the safety rail to rotate down to its closed position.