

PRODUCT & ASSEMBLY GUIDE

Fixed

FOUNDATION




ELECTRIC



ARTICULATION



HI-LO

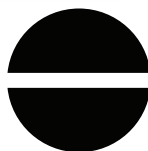


FIXED



MANUAL


The Fixed Foundation is stationery. A Box Spring or Bunkie Board are installed beneath the mattress for different safety rail heights.

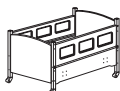

☐ **SleepSafe (BASIC)**



☐ **SleepSafe (Low Bed)**

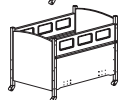

☐ **SleepSafe II (Medium Bed)**



☐ **SleepSafer (Tall Bed)**

☐ Dual View



☐ **SleepSafer with Extension**

☐ Dual View ☐ with Topper



SleepSafeBed.com | 866.852.2337

3629 Reed Creek Drive | Bassett, VA 24055 | FDA Registered

Table of Contents

1. Pre-Assembly Information
2. Component Chart #1
3. Component Chart #2
4. Assembly Guide: SleepSafe BASIC & Low
5. SleepSafe II Medium Bed
6. Assembly Guide: SleepSafer Tall Bed
7. SleepSafer Tall Bed Dual View
8. Assembly Guide: SleepSafer Tall Bed
9. with Extension / Top
10. Safety Rail Adjustment Instructions
11. Padding Information
12. Mattress Information
13. Final Installation Checklist
14. Operator's Guide
15. Maintenance / Popular FAQ's
16. Warranty Information
17. DME Provider / Technical Support
18. Serial Number

This guide is available as a PDF. Contact Customer Service at 866.852.2337 for more information.



FIXED FOUNDATION

Tools Needed for Assembly




Allen and Torx Wrenches
(Supplied)



Flat and Phillips Head
Screwdrivers



3/8" Open or Closed
End Wrench

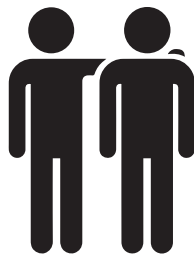

Two People for Assembly

Note! Read Before Assembly

- **KEEP THIS PRODUCT & ASSEMBLY GUIDE.**

IT CONTAINS THE SERIAL NUMBER

- CAREFULLY REMOVE AND LAY OUT ALL COMPONENTS. FAMILIARIZE YOURSELF WITH INSTRUCTIONS TO FACILITATE THE ASSEMBLY OF YOUR BED.

- CHECK COMPONENTS AND CONTENTS OF ENCLOSED HARDWARE BAG WITH THE HARDWARE LIST ON FOLLOWING PAGES.

- KEEP THE FOUR CARDBOARD STANDS FOR ASSEMBLY AND POSSIBLE FUTURE USE.

DUE TO THE PRESENCE OF SMALL PARTS DURING ASSEMBLY, KEEP OUT OF REACH OF CHILDREN.

- IMPORTANT: FOLLOW THE INSTRUCTIONS IN THE SPECIFIED ORDER TO ENSURE THE BED IS PROPERLY ASSEMBLED.

- COMPLETE ASSEMBLY. ENSURE EACH COMPONENT IS INSTALLED. DO NOT OMIT ANY COMPONENT FROM INSTALLATION.

- NEVER USE THIS BED IF THERE ARE ANY MISSING FASTENERS, LOOSE JOINTS, BROKEN PARTS, TORN MESH, OR PADDING INADEQUACIES. CONTACT SLEEPSAFE BEDS AT 866.852.2337 FOR ASSISTANCE.

- NEVER SUBSTITUTE OR ALTER PARTS, THE PRODUCT INCLUDING SIDE RAILS, MUST BE FULLY INSTALLED PRIOR TO USE.

- IF BED IS PADDED, CHECK THAT ALL PADDING IS FIRMLY ATTACHED.

- DO NOT USE POWER TOOLS DURING ASSEMBLY OR DISASSEMBLY

- WARNING STRANGULATION HAZARD: STRINGS CAN CAUSE STRANGULATION! NEVER PLACE BED NEAR WINDOW WHERE CORDS FROM BLINDS OR DRAPES MAY STRANGLE A CHILD.

- KEEP REMOTE CONTROL CORD AWAY FROM BED WHILE MOVING THE SAFETY RAIL UP OR DOWN TO PREVENT PINCHING OF THE CORD.

- KEEP REMOTE CONTROL CORD AWAY FROM BED'S INTERIOR TO PREVENT ENTANGLEMENT AND STRANGULATION

- AN UNINTERRUPTED POWER SUPPLY (UPS) UNIT IS RECOMMENDED FOR MISSION CRITICAL CONTROL OF BED FOR USER










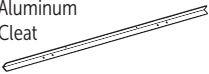

- DO NOT SUBSTITUTE MATTRESS. IT IS SPECIFICALLY DESIGNED AND FITTED FOR A SLEEPSAFE® BED AND ITS USER.














SleepSafeBed.com | 866.852.2337

3629 Reed Creek Drive | Bassett, VA 24055 | FDA Registered

Included Components for Assembly

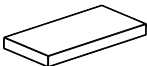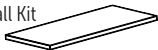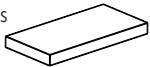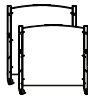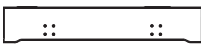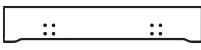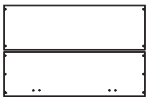	SLEEPSAFE (Basic & Low)	SLEEPSAFE II (Medium Bed)	SLEEPSAFER (Tall Bed)	SLEEPSAFER (w/Extension)	SLEEPSAFER (w/Ext. & Top)	SSR (Dual View w/Extension)	SSR (Dual View w/Ext. & Top)
Aluminum Cleat 	2	2	2	2	2	2	2
Cross Support for Box Spring or Bunkie Board 	2	2	2	2	2	2	2
25 mm Allen Bolt 	4	4	4	4	4	4	4
90 mm Allen Bolt 	8	12	16	28	28	12	28
3/4" Hex Bolt 	8	8	8	8	8	8	8
Barrel Nut 	8	12	16	28	40	12	40
40 mm Allen Bolt 					Tw-12 Fu-18 Qu-18		Tw-12 Fu-18 Qu-18
Allen / Torx Wrench Set (Various sizes) 	1	1	1	1	1	1	1
Square Bit Drive 	1	1	1	1	1	1	1
Casters (Attached) 	4	4	4	4	4	4	4
Extension Safety Latch & 2 Screws 			1	1		2	2

Included Components for Assembly

	SLEEPSAFE (BASIC & Low)	SLEEPSAFE II (Medium Bed)	SLEEPSAFER (Tall Bed)	SLEEPSAFER (w/Extension)	SLEEPSAFER (w/Ext. & Top)	SLEEPSAFER (Dual View)	SSR (Dual View w/Extension)	SSR (Dual View w/Ext. & Top)
Box Spring 	1	1	1	1	1	1	1	1
OR								
Bunkie Board & Install Kit 	1	1	1	1	1	1	1	1
Mattress 	1	1	1	1	1	1	1	1
Headboard and Footboard Set 	1	1	1	1	1	1	1	1
Side Panel 	2	2						
Front Panel 			1	1	1	2	2	2
Back Panel 			2	2	2			
Safety Rail 	2	2	2	2	2	4		4
Extension End Panel 				2	2		2	2
Extension Side Panel 				2	2		2	2
Extension Topper 					1			1

Assembly Guide

✓ SleepSafe (BASIC & Low)

✓ SleepSafe II (Medium Bed)

After unpacking bed and checking the contents

1

Are you installing a Bunkie Board or a Box Spring?

IMPORTANT!

The SleepSafe (Low Bed) and SleepSafe II (Medium Bed) have different panel configurations for installing the aluminum cleats.



SleepSafe (Low Bed)
Bunkie Board
or Boxed Spring



SleepSafe II
(Medium Bed)
Bunkie Board





SleepSafe II
(Medium Bed)
Boxed Spring

2

Prepare Side Panels:

Attach Aluminum Cleat to each Panel.

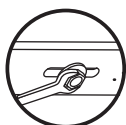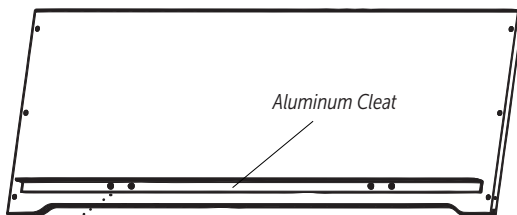


(Use 3/8" wrench to install eight 3/4" Hex Bolts. Four on each Side Panel)

3

Insert Barrel Bolts into Side Panels.

Align and push in Barrel Nut as shown. Use flathead screwdriver for assistance.

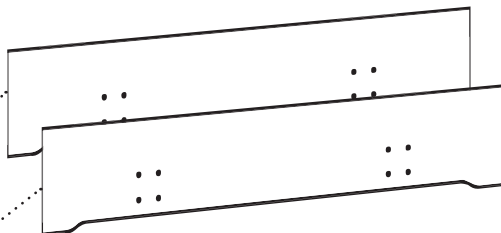

*SleepSafe Bed (Low Bed) Side Panels use 8 Barrel Bolts
SleepSafe II (Medium Bed) Side Panels use 12 Barrel Bolts*



FIXED FOUNDATION



SleepSafeBed.com

1-866-852-2337


3629 Reed Creek Dr. | Bassett, VA 24055

4


Attach Headboard and Footboards to Side Panels.

Attach Cross Supports to Aluminum Cleats.

(Use 4mm Allen Wrench to fasten all Allen Bolts)

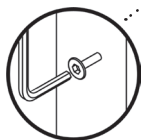


NOTE: For Full and Queen size beds, use provided bed slats for installation between the aluminum cleats.

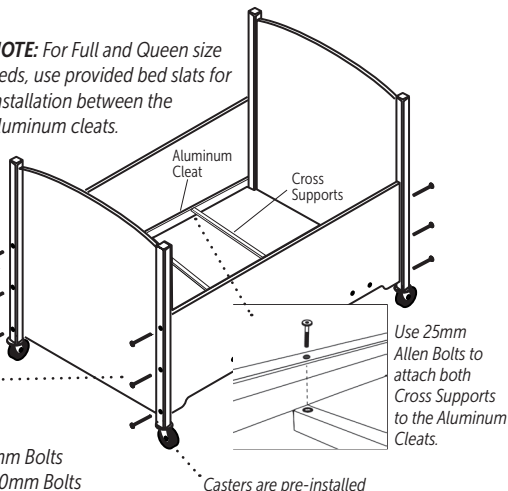


SleepSafe Bed (Low Bed) uses (8) 90mm Bolts
SleepSafe II (Medium Bed) uses (12) 90mm Bolts

5

Prepare Safety Rail Hinge Pins for Installation.

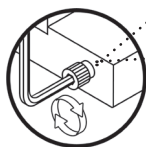

CLOCKWISE
INCREASES play.
COUNTER-CLOCKWISE
DECREASES play.

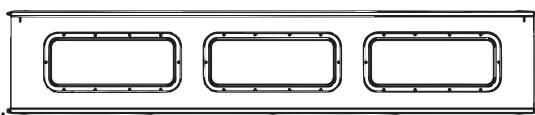

Factory preset has hinge pins that may be extended. These pins need to be adjusted evenly on both sides to fit properly when inserted into the posts.

NOTE: Full instructions for safety rail adjustment are found on page 10.



6

Install Safety Rails, Bunkie Board, or Box Spring, then Mattress.


Insert the safety rail hinge pins into both posts when safety rail is in the down position.

Strike

Hinge Pin

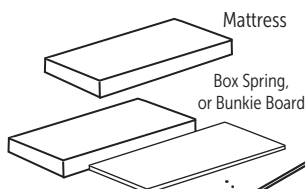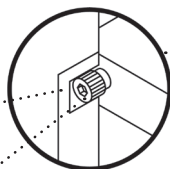

Same rail installation for both sides of the bed.

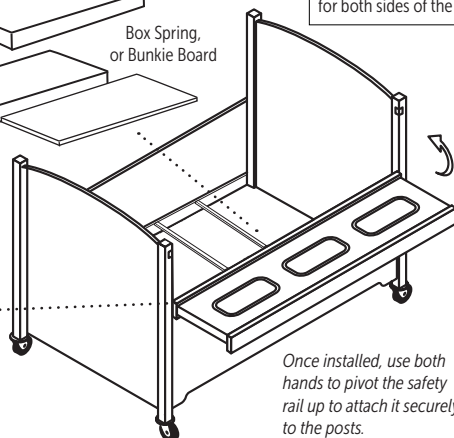

Once installed, use both hands to pivot the safety rail up to attach it securely to the posts.

Assembly Guide

☒ SleepSafer (Tall Bed)


☒ SleepSafer (Dual View)

After unpacking bed and checking the contents


1 Are you installing a Bunkie Board or a Box Spring?

IMPORTANT!

Be sure to check the correct configuration for Bunkie Board or Box Spring and desired location for installing the aluminum cleats.


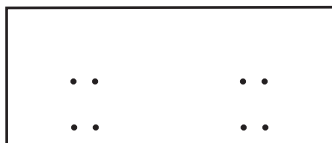


SleepSafer (Tall Bed)
BACK PANEL
Bunkie Board
or Boxed Spring

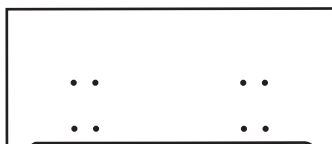

SleepSafer (Tall Bed)
FRONT PANEL
Bunkie Board
or Boxed Spring.
NOTE: Use 2 Front
Panels for Dual View



2 Prepare Side Panels: Attach Aluminum Cleat to each Panel.

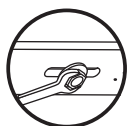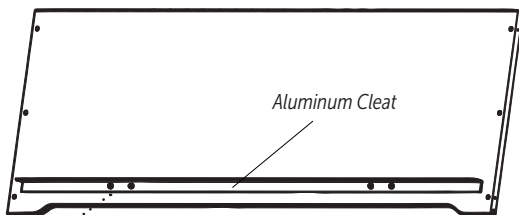

(Use 3/8" wrench to install eight 3/4" Hex Bolts. Four on each Side Panel)

3 Insert Barrel Bolts into Back and Front Panels.

Align and push in Barrel Nut as shown. Use flathead screwdriver for assistance.

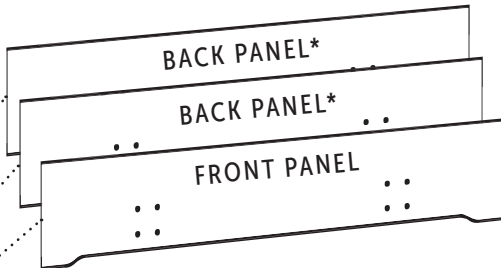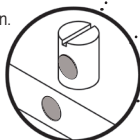

SleepSafer Bed (Tall Bed) Back Panels uses 10 Barrel Bolts
SleepSafer Bed (Tall Bed) Front Panel use 6 Barrel Bolts

* Dual View Option uses 2 Front panels only. No Back Panels.



FIXED FOUNDATION



SleepSafeBed.com

1-866-852-2337


3629 Reed Creek Dr. | Bassett, VA 24055

4 Attach Headboard and Footboards to Side Panels.

Attach Cross Supports to Aluminum Cleats.

(Use 4mm Allen Wrench to fasten all Allen Bolts)

NOTE: For Full and Queen size beds, use provided bed slats for installation between the aluminum cleats.

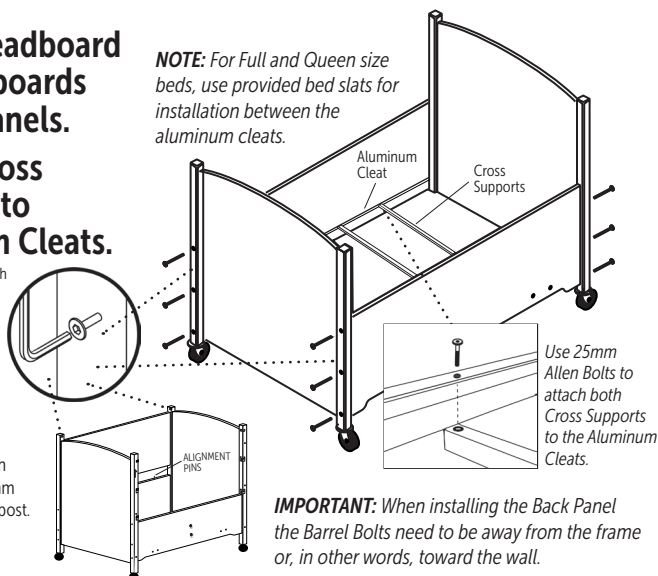

5 Install Back Panel.


Set in place with alignment pins, then fasten with (2) 90mm Allen Bolts to each post.

NOTE: Not necessary for Dual View option

IMPORTANT: When installing the Back Panel the Barrel Bolts need to be away from the frame or, in other words, toward the wall.

6 Prepare Safety Rail Hinge Pins for Installation.

CLOCKWISE
INCREASES play.
COUNTER-CLOCKWISE
DECREASES play.

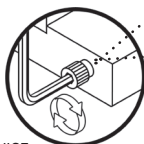

Factory preset has hinge pins that may be extended. These pins need to be adjusted evenly on both sides to fit properly when inserted into the posts.

NOTE: Full instructions for safety rail adjustment are found on page 10.



7 Install Safety Rails, Bunkie Board, or Box Spring, then Mattress.

Insert the safety rail hinge pins into both posts when safety rail is in the down position.

Mattress

Box Spring, or Bunkie Board

Install Four Rails for DUAL VIEW option.

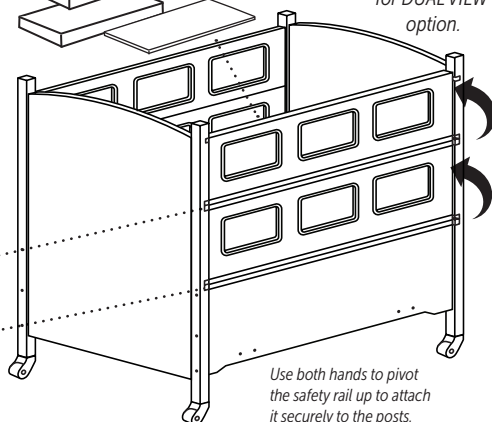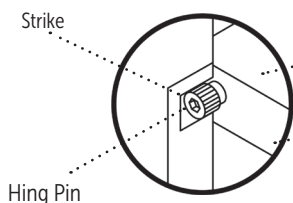

Use both hands to pivot the safety rail up to attach it securely to the posts.

Assembly Guide


✓ **SleepSafer (Tall Bed)**
Open & Closed Extensions

After unpacking bed and checking the contents

1

Set Extension Side Rails Upside Down and on their Legs.


Assemble this with two people,
otherwise one person can
assemble one panel at a time
after the the rest of the bed
is assembled.

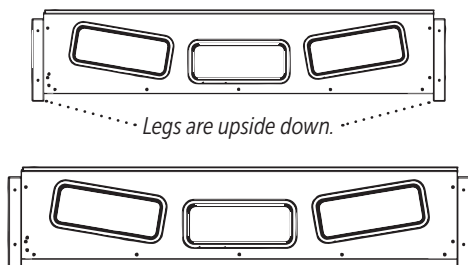

2

Attach Extension Ends to Side Rails. Flip Assembly Over.


Illustration shows
assembly upside down.
Flip unit over when
assembly completed.

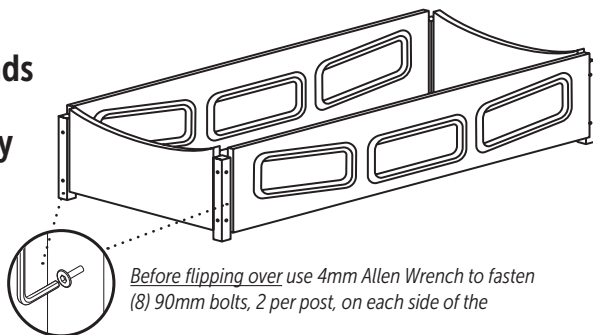

3

If Applicable, Attach the Top to the Extension.

Use 4mm Allen
Wrench to attach
Extension Top
with (12) 40mm
Allen Bolts.

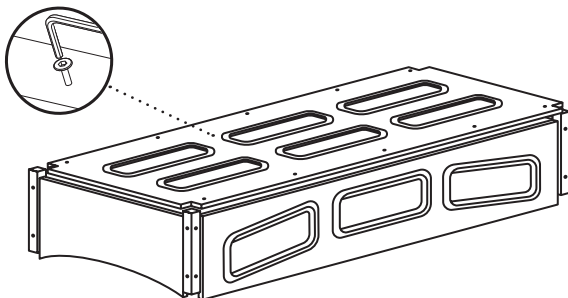

FIXED FOUNDATION



SleepSafeBed.com
1-866-852-2337

3629 Reed Creek Dr. | Bassett, VA 24055

4

This SleepSafer® Tall Bed Extension installation step is based on ordering it with the SleepSafer® Tall Bed. Retrofitting this Extension requires additional instruction.

A Loosen top bolts on all 4 bed posts.




This allows play in the frame to fit the Extension.

B Set the Extension on top of bed posts.

Extension posts should align with the bed posts.

C Fasten Extension to the Bed.

Fasten (1) 90mm Allen Bolt into each post to attach Extension. Tighten all bolts.

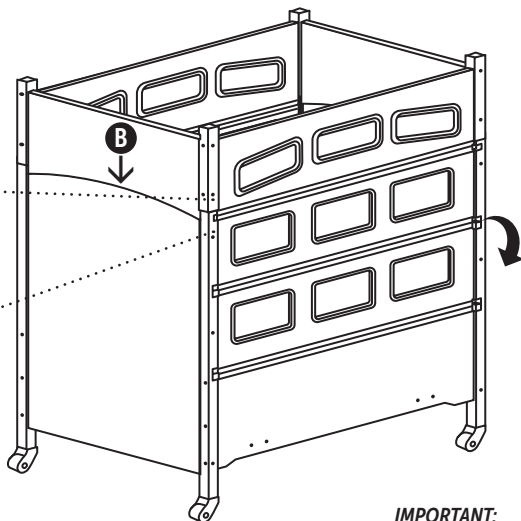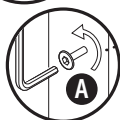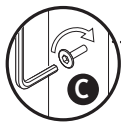

IMPORTANT:



Bottom Safety Rail pivots down.
Use both hands to carefully operate.

5

Install Safety Latch to Extension.

Align safety latch to threaded inserts located in the predrilled holes in the Extension. Use Phillips head screwdriver to mount the latch to the Extension.

The latch will catch and secure the top safety rail when raised into position. Use thumb to push latch up to release the rail.

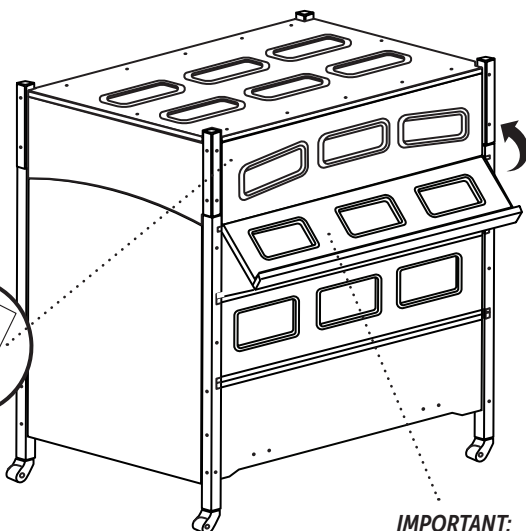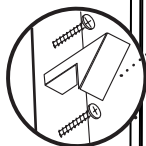

IMPORTANT:

Be sure to support safety rail with both hands while moving the rail into their secure positions.

Up and Down Safety Rail Adjustment

1 Does the Safety Rail need adjustment?

For proper operation the pin should be centered in the strike. If not centered, follow the following steps to center the pin in the strike.

2 Remove Strike from Bed Post.

Use Drill with Torx bit to remove screws from the Strike. Gently tap the strike out of the bed post with hammer or mallet to access



3 Adjust Set Screw.

Use Torx 10 wrench to adjust set screw. Keep in mind which direction that you need to adjust the pin. If the pin needs to be raised, turn the set screw clockwise. If the pin needs to be lowered, turn the set counter-clockwise.

4 Reinsert Strike into Bed Post.

Check to see if the Pins are centered properly within the Strikes.

Additional Tools Needed:
Hammer or Mallet
Portable Electric Drill

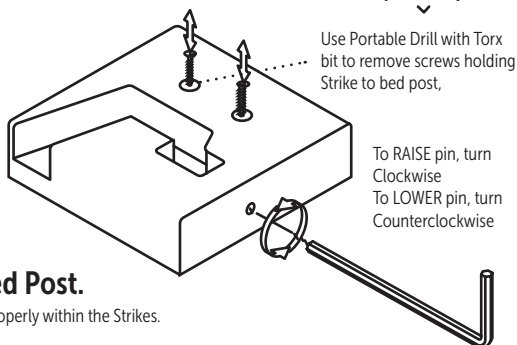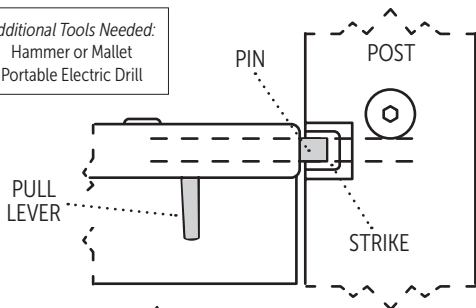


Side by Side Safety Rail Adjustment

1 Does the Safety Rail need adjustment?

When up and secure, check the spacing between the end of the safety rail and the headboard and footboard legs. They should be spaced evenly.

2 Remove Safety Rail and adjust the play.

If spacing is not even, or there is too much side-to-side play in the rail, turn the black hex bolt using the 1/4" Allen Wrench provided.

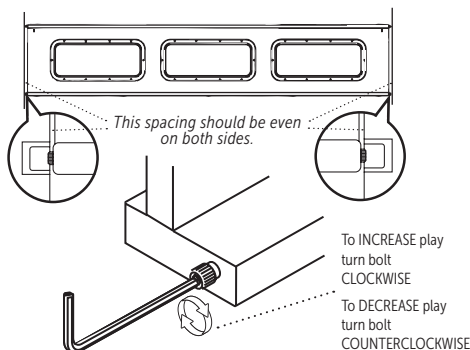

FIXED FOUNDATION



SleepSafeBed.com

1-866-852-2337

3629 Reed Creek Dr. | Bassett, VA 24055

SleepSafe Bed Padding Information

Materials:

- 70 Denier Nylon Fabric Cover with a Fluid Resistant Healthcare Coating
- 3/4" High Density Foam Padding

Padding Features:

The safety rail pads extend up and over the rails and attach to the outside of the safety rails. The headboard and footboard pads extend to the top and to the edges of the legs of headboard and footboard.

- Anti-Static
- Bacteria Resistant
- Malodor Protection
- Fluid Resistant
- Fungal Resistant
- Flame Retardant
- Tear Resistant



Padding Attachment:


- Industrial strength Velcro is used to secure the padding to inside panels and safety rails.

Padding Maintenance


- For cleaning, use warm water and a mild cleanser. If the padding is over the windows, it can be removed and cleaned. If the padding is around the windows, please do not remove. It is best to clean this padding in its place.

Important Note:


Padding is installed at the factory and will cover ALL 4 sides. Padding only 1 or 2 sides is NOT an option. Padding removal will cause a gap between the mattress and the bed's interior.

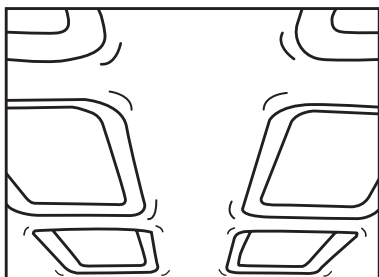


Padding for a SleepSafer Tall Bed with an Extension Top works around the screen mesh windows for better ventilation.

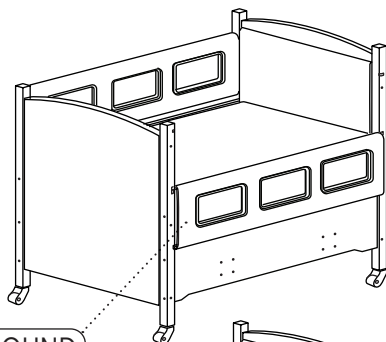


AROUND

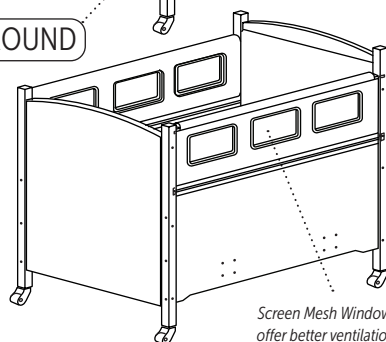

Screen Mesh Windows offer better ventilation.

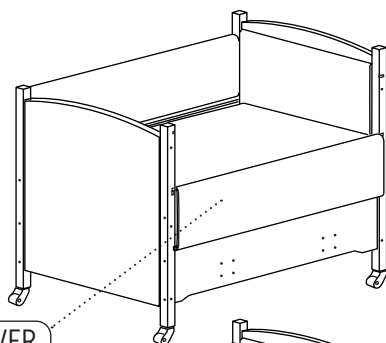

OVER

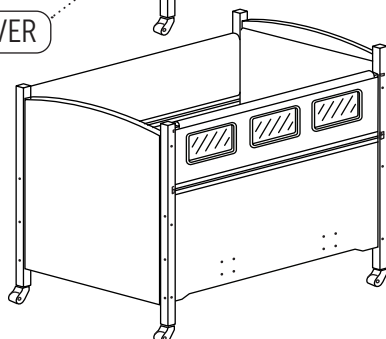

SleepSafe Bed Mattress Information


SleepSafe Bed® – Our Memory Foam Mattress with Gel Technology

SleepSafe® Bed mattress support surfaces provides optimized pressure redistribution, rapid heat dissipation, and increases in breathability compared to standard foam.

- Reduced incidence of pressure ulcers by channeling heat away from the body much faster than standard foam, virtually ridding perspiration moisture.
- Superior pressure redistribution achieved by providing more support and less cradling at higher levels of compression, distributing pressure over a larger surface area and eliminating static fatigue.

How Does It Work?

The technology works by allowing air to pass through open cells, channeling it away from the body instead of trapping it within the closed cells of traditional memory foam. The infusion of gel polymers throughout the foam contributes both cooling properties and the virtual elimination of static fatigue. As a result of the marriage between open cell foam and gel infusion, the mattress holds its shape better, redistributes pressure over a larger surface area, and successfully wicks heat away from the body, leaving patients cool, dry, and comfortable.

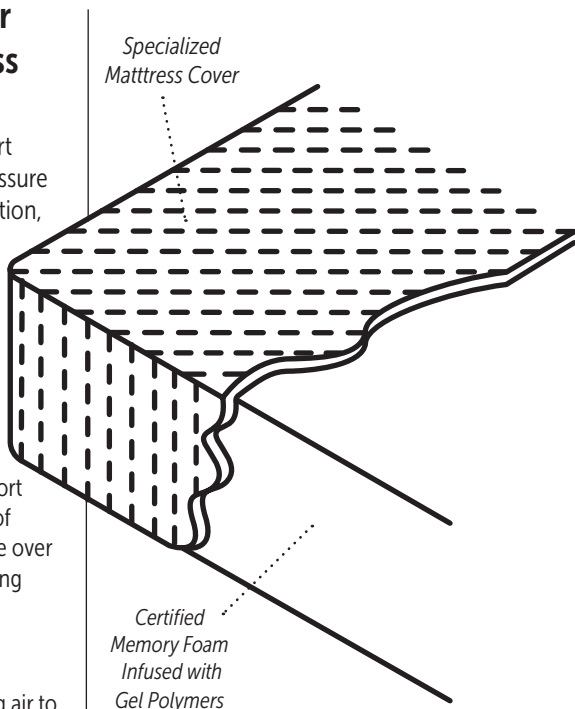

Foam products meet the following criteria:

- | | |
|-------------------------|--------------------------------------|
| • Low Emissions (VOC's) | • Performance – Tested |
| • BPA Phthalate – Free | • Made without Ozone Depleters |
| • CFC – Free | • Mercury, Lead and Heavy Metal Free |
| • BHT – Free | • Made without Formaldehyde |
| • PBDE – Free | • Methylene Chloride – Free |
| • Emissions – Tested | • Environmentally Innovative |
| • Content – Tested | • Material Certification |

Other notes about the foam in mattresses:

- Foam products use a blend of natural oil-based and conventional polyols.
- Benefits farmers looking for a new market for crops.
- Polyol is made partially from plant-based renewable oils such as soy bean and castor beans.



FIXED FOUNDATION



SleepSafeBed.com


1-866-852-2337

3629 Reed Creek Dr. | Bassett, VA 24055

Final Bed Installation Checklist


1 Check all bolts to see that they are tight.

Be sure all fasteners that hold the components are hand tightened with wrenchs.

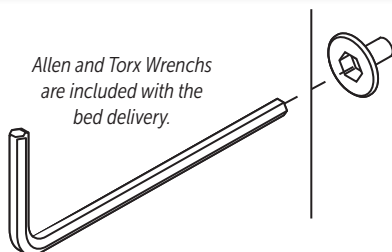


2 Check Safety Rails to be sure they operate and latch properly.

Safety Rails should pivot correctly, align properly and securely latch into the gate located on the bed posts.

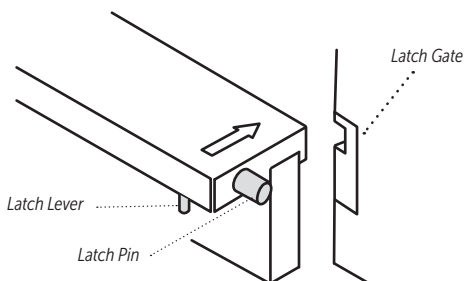


3 Check fit between headboard, footboard, bed sides and mattress.

The mattress should always fit snugly against the sides. If not, contact the DME or Customer Service at SleepSafe Beds.

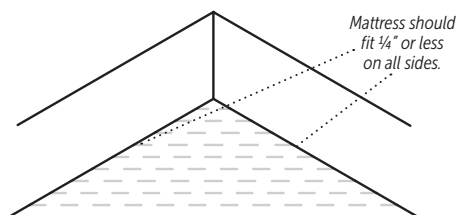

4 For Extensions, check operation of Safety Latch holding the top Safety Rail.

Top Safety Rail should set in place when lifted up and into place on the Extension.

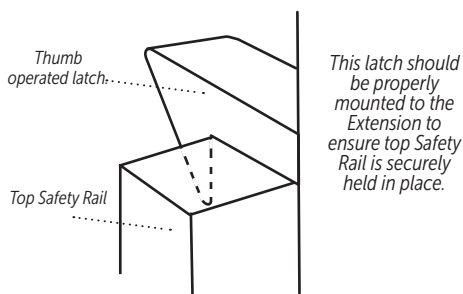

Operator's Guide: Fixed Foundation


Handling, operating and securing the Safety Rails.

Safety Rails on the SleepSafe (Low Bed), SleepSafe II (Medium Bed) and SleepSafer (Tall Bed) are removable. Removing the Top Rail first on the Tall bed is recommended.

The top Safety Rail on the SleepSafer with the Extension is not removable. The Top Rail attaches to the Extension.

It is always recommended to handle the Safety Rails gently and firmly with both hands. Be sure both sides of the Safety Rail are securely attached to the post before leaving the bed alone.

REMOVE Safety Rails from their down position. Gently lift the rail UP and OUT. To ATTACH, place IN both strikes and let the rail DOWN.

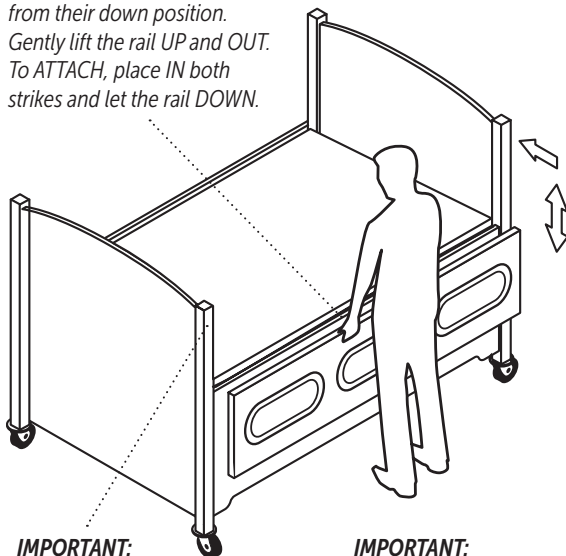


IMPORTANT:
BE SURE BOTH SIDES OF THE SAFETY RAILS ARE SECURED TO THE POSTS.

IMPORTANT:
USE BOTH HANDS WHEN OPERATING SAFETY RAILS.


How the Casters Work.

The casters are pre-installed at the factory in the bottom of the headboard and footboard legs. It is recommended that the casters are locked when the bed is in use to add to the bed's stability.

The casters are locked when the tab is in the down position. Unlocked when the tab is up.

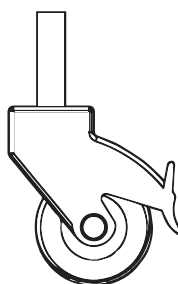

LOCKED

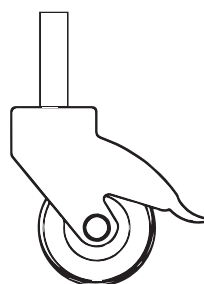

UNLOCKED



FIXED FOUNDATION



SleepSafeBed.com

1-866-852-2337

3629 Reed Creek Dr. | Bassett, VA 24055

Frequently Asked Questions

How do I get replacement parts? What should I do if I have a repair problem?

Please call your local dealer that helped you purchase the bed. They should be able to help with any problem you may have. If your original dealer cannot help you with replacement parts or repair, please contact us directly so we may assist you at 866-852-2337.

My child's medical needs have changed. Can I change the foundation?

Yes. Most SleepSafe® Bed foundations can be converted to another bed. We will be glad to assist you. Give us a call, toll-free, at 866-852-2337 for more information about SleepSafe® Bed conversions. To convert any SleepSafe Bed foundation, you will need to purchase a Conversion Kit from SleepSafe® Beds. Insurance may or may not cover the additional cost of the conversion kit. It may be necessary to submit another letter of medical necessity justifying the change. You will need to make the purchase through your local DME provider of SleepSafe® Beds.

Can I buy just a mattress?

Yes, as a replacement for a SleepSafe® Bed mattress. We do not recommend using a SleepSafe® Bed mattress for another bed type. Our mattresses are designed and custom sized to fit inside a SleepSafe® Bed and may not be appropriate for other beds. Safety is our number one priority and our mattresses are designed to fit snugly within the wooden bed frame to reduce the chance of burrowing or entrapment.

How do I clean my SleepSafe® Bed?

All surfaces of a SleepSafe® Bed can be cleaned with water and mild detergent using a soft cloth. Please avoid any abrasives and solvents.

Annual Maintenance Checklist

- ☒ ***Tighten all bolts on panels and posts.***
Use provided Torx and Allen Wrenches to hand tighten. Avoid power tools.
- ☒ ***Lubricate working components beneath the bed.***
Lubricate the pins, actuators and hinges.
- ☒ ***Clean Padding and Check Velcro® if bed is padded.***
Use a mild detergent to gently clean surfaces, avoid abrasive cleaning pads or sponges. Check to see that Velcro® is attached properly.

Bed/Mattress Warranty Information

Full One-Year SleepSafe® Bed Warranty

The SleepSafe® Bed is warranted against defects in workmanship or materials for a period of one year from the date of purchase. Sleep Safe Beds, LLC will repair or replace any defective part at no cost to the purchaser if the shipping of the part is within the continental United States.

Limited Five Year SleepSafe® Bed Warranty

During the second through fifth years from the date of purchase, SleepSafe Beds, LLC will replace any part found to be defective. Purchaser shall pay all service and shipping costs related to the replacement of the defective part.

Terms and Conditions for SleepSafe® Bed Warranty

A SleepSafe® Bed is expressly prescribed for a single user. This warranty will cover the bed for this user only. Warranty coverage beyond this use is handled on a case by case basis. This warranty applies to normal use and does not cover any damage caused by excessive wear, abuse, misuse, mishandling or modification of the product.

SleepSafe® Bed Mattress Warranty Information

All SleepSafe® Bed models are shipped with a CertiPUR-US® memory foam mattress with gel infused technology. This support surface provides optimized pressure redistribution rapid heat dissipation and increases in breathability compared to standard foam. Memory foam infused with gel polymers is designed for patient comfort as well as clinical superiority. Benefits include: reduced incidence of pressure ulcers by channeling heat away from the body significantly faster than standard foam, virtually eliminating perspiration moisture. The mattresses are water resistant, but are not waterproof.

Full One-Year Warranty for SleepSafe® Bed Mattress

Full one year warranty from date of purchase. SleepSafe® Beds, LLC will replace the mattress due to a defect in the materials and workmanship of the mattress, at no cost to the purchaser.

Limited Five Year Warranty for SleepSafe® Bed Mattress

During the second to fifth year, SleepSafe® Beds, LLC will replace the mattress due to a defect in the materials or workmanship of the mattress, with the shipping costs to be paid by purchaser.

Terms and Conditions for SleepSafe® Bed Mattress Warranty

Any breakdowns or deterioration of the product caused by factors other than a defect in the materials and workmanship of the mattress, such as repeated exposures to bodily fluids by incontinence, are not covered by this warranty.



FIXED FOUNDATION



SleepSafeBed.com

1-866-852-2337

3629 Reed Creek Dr. | Bassett, VA 24055

Durable Medical Equipment Provider

Bed Serial # _____

Installation Date _____

DME Provider _____

DME Contact _____

DME Address _____

DME Phone _____

DME Email _____

Installed by _____

Customer Service & Technical Support

SleepSafe Bed Support

Hours: 8 AM to 5 PM / M-F / EST

Toll Free: 866.852.2337

Local: 276.627.0088

Fax: 276.627.0234

Web: SleepSafeBed.com

In case of an EMERGENCY after hours, please call 866-852-2337 for instructions on how to access the on-call help desk.

DME Provider Support


DME: _____

Contact: _____

Phone: _____

Email: _____

Web: _____

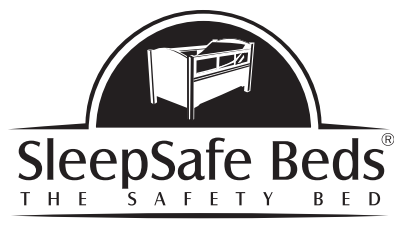

SleepSafeBed.com | 866.852.2337

3629 Reed Creek Drive | Bassett, VA 24055 | FDA Registered

WI-820.120-1 Fixed [1] Rev. 01