

# ASSEMBLY UPDATE!

## SleepSafer® Bed Extension Latch FASTENER REPLACEMENT KIT


*This Kit is for replacing the existing latch that holds the upper safety rail in place when lifted up to allow access to the SleepSafer® Bed with an Extension.*



SleepSafe Beds, LLC  
3629 Reed Creek Drive  
Bassett, VA 24055

Toll Free: 866.852.2337  
Fax: 276.627.0234  
SleepSafeBed.com  
sleepsafebed@sleepsafebed.com


1

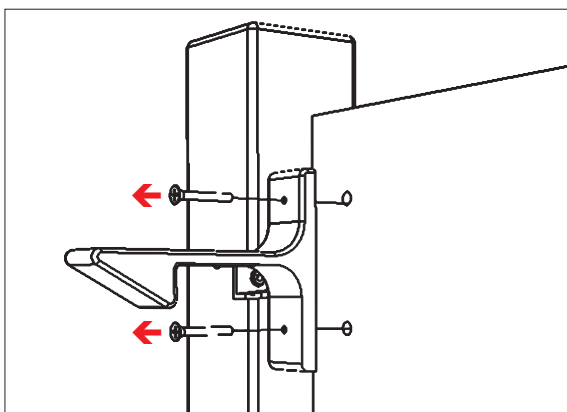

### REMOVE EXISTING LATCH

On the the SleepSafer® Bed Extension, locate the latch. Use a phillips head screwdriver to remove the two screws from the threaded inserts that are imbedded into the wood.


2

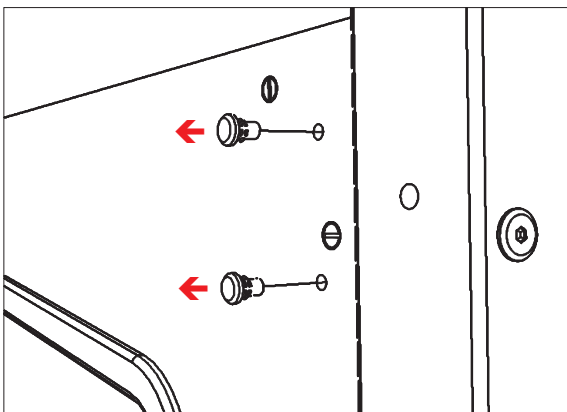

### REMOVE THREADED INSERTS

On the the inside of the SleepSafer® Bed Extension, locate the threaded inserts. Use a hammer and a small punch tool to gently tap the inserts from the wood. Be careful not to disturb the opening as you will need to replace these inserts.

3

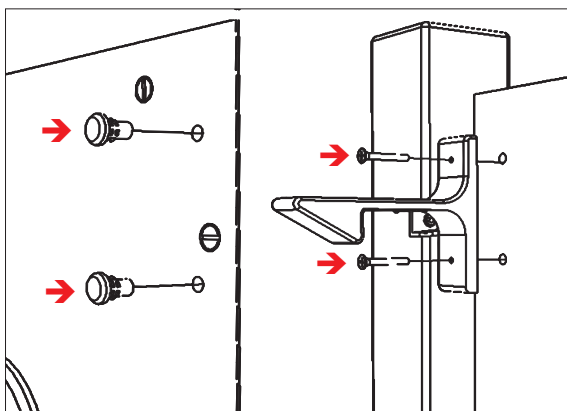

### REPLACE THREADED INSERTS AND LATCH.

Carefully insert and gently tap the replacement inserts into the existing holes. Fasten the latch with the replacement screws. Be careful not over tighten.