

# RETROFIT KIT Dreama™ by Jenx

## How to install a Dreama™ System into a SleepSafe® Bed

**Includes:** Dreama™ 24hr Positioning System; Deck Pad; Deck Strap; Deck Screws

**Tools Needed:** 4mm Allen Wrench (Included) or Drill with 4mm Allen Bit




3629 Reed Creek Dr. / Bassett, VA 24055

Toll Free: 866.852.2337

SleepSafeBed.com

sleepsafebed@sleepsafebed.com

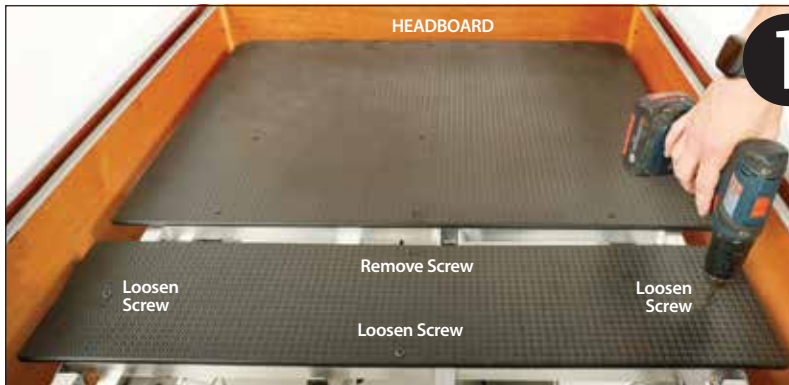

**1**

**LOWER SAFETY RAILS. REMOVE MATTRESS. LOOSEN 3 SCREWS, REMOVE 1 IN THE 2ND BED DECK PANEL.** With 4mm Allen Wrench (or drill with bit), loosen 3 screws and remove 1 in the deck panel as shown. The purpose is to allow room to feed the strap beneath the deck.



**2**

**FEED THE STRAP BENEATH BOTH SIDES OF THE DECK PANEL.** Feed the strap beneath the deck. The buckle should line up with the loosened screw. Once buckle in place, replace removed screw, and then tighten all the screws.



**3**

**PLACE PAD ON TOP OF DECK INSIDE THE SLEEPSAFE BED.**




Note the position of the strap buckle once the pad is placed on top of the deck.




## 4 UNROLL THE DREAMA SYSTEM ON TO THE PAD AND FIT INTO BED.

The Dreama™ System base is rolled up and needs to be unrolled and placed on the pad inside the bed. Also, locate the holes in the base where the strap will be threaded. See below.

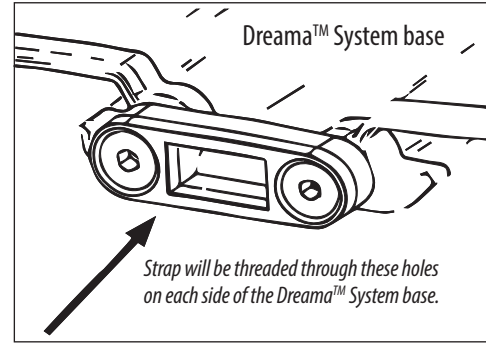

## 5 THREAD STRAP THROUGH BASE OF DREAMA™ SYSTEM. THEN TIGHTEN STRAP.

Thread the strap through the holes in the 8th row on each side of the Dreama™ System base. Once in place and the buckle is securely tightened, the Dreama™ System bolsters can be installed.

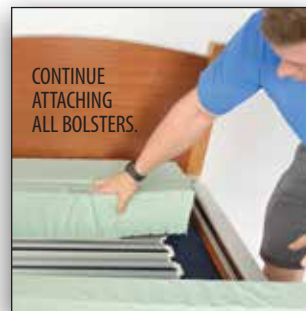

## 6 ATTACH ALL BOLSTERS TO BASE. THE TWO WIDER BOLSTERS GO ON EACH END.

Each bolster attaches by inserting their plastic tabs into the Dreama™ System base. Be sure to attach each side. Start at the head with a wider bolster, continue with narrow bolsters, and finish with the other wider bolster.

**\*INSTALLATION NOTES:** For beds with fixed foundations, use the same process but wrap the strap around the bunkie board or box spring.