

ASSEMBLY DETAILS

SleepSafe® Bed LATCH SAFETY BLOCK - ADD ON

This kit is for installing a latch safety block.


Kit includes: 4 latch safety blocks and 4 phillips head screws.

Tools needed: Phillips head screwdriver



SleepSafe Beds, LLC
3629 Reed Creek Drive
Bassett, VA 24055

Toll Free: 866.852.2337
Fax: 276.627.0234
SleepSafeBed.com
sleepsafebed@sleepsafebed.com

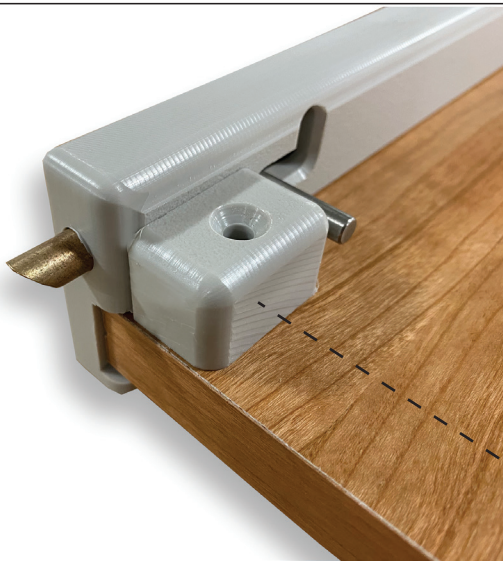

This kit is designed as an add-on option for beds whose occupants can reach over the top of the safety rail and pull against the latch handle, causing it to disengage. The blocks help deter an accidental unlatching of the mechanism.

1

REMOVE SAFETY RAIL FROM BED

Lay it down so the latch side is facing up.

2

ATTACH BLOCK TO SAFETY RAIL

Place the latch block directly under the safety rail cap as shown and insert screw into the hole. Tighten the screw with your phillips head screwdriver until there is no play between the block and the safety rail.

3

REPEAT ABOVE STEPS TO THE ALL SIDES OF THE SAFETY RAILS

Once completed, reinstall safety rails and confirm the latch operates properly. Block should provide difficulty in latch operation.