

# ASSEMBLY DETAILS

## SleepSafe® Bed IV POLE IV POLE ADD-ON KIT

This kit is for installing an IV Pole to a SleepSafe® Bed.

*Kit includes:* IV Pole; IV Ram Hook Top; 2 Brackets; 4 screws.

*Tools needed:* T-10 Torx wrench (provided); Phillips head screwdriver.



SleepSafe Beds, LLC  
3629 Reed Creek Drive  
Bassett, VA 24055

Toll Free: 866.852.2337  
Fax: 276.627.0234  
SleepSafeBed.com  
sleepsafebed@sleepsafebed.com



1

### ASSEMBLE BRACKETS TO POLE

The IV Pole will sit in the closed-end bottom bracket. For proper support of the IV Pole, the top bracket needs to be attached at least 6" from the bottom bracket. Use the T-10 Torx wrench to loosen the set screw to adjust the top bracket and then tighten the set screw to secure it.

2

### DETERMINE WHERE THE IV POLE WILL BE LOCATED

The IV Pole must be installed on the OUTSIDE of the headboard or footboard. Take the time necessary to consider the benefits of proper IV Pole placement.

3

### ATTACH BRACKET TO BED AND FOLD DOWN RAM HOOKS

Attach lower bracket to bed first using two screws making sure to keep the IV Pole straight up. Once lower bracket is in place attach upper bracket using the two remaining screws.

