

ASSEMBLY DETAILS

SleepSafe® Bed IV POLE IV POLE ADD-ON KIT

This kit is for installing an IV Pole to a SleepSafe® Bed.

Kit includes: IV Pole; IV Ram Hook Top; 2 Brackets; 4 screws.

Tools needed: 3/32 Allen Wrench (provided); Phillips head screwdriver.



SleepSafe Beds, LLC
3629 Reed Creek Drive
Bassett, VA 24055

Toll Free: 866.852.2337
Fax: 276.627.0234
SleepSafeBed.com
sleepsafebed@sleepsafebed.com



1

ASSEMBLE BRACKETS TO POLE

The IV Pole will sit in the closed-end bottom bracket. For proper support of the IV Pole, the top bracket needs to be attached at least 6" from the bottom bracket. Use the 3/32 allen wrench to loosen the set screw to adjust the top bracket and then tighten the set screw to

2

DETERMINE WHERE THE IV POLE WILL BE LOCATED

The IV Pole must be installed on the OUTSIDE of the headboard or footboard. Take the time necessary to consider the benefits of proper IV Pole placement.

3

ATTACH BRACKET TO BED AND FOLD DOWN RAM HOOKS

Attach lower bracket to bed first using two screws making sure to keep the IV Pole straight up. Once lower bracket is in place attach upper bracket using the two remaining screws.

