

PRODUCT & ASSEMBLY GUIDE

# InSIGHT™ Safety Bed

FIXED FOUNDATION (TWIN)




ELECTRIC



ARTICULATION



HI-LO

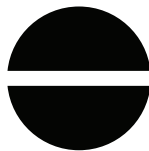


FIXED



MANUAL

*The Fixed Foundation is stationary. The Bunkie Board is sewn together with the mattress as a combination to address the chance of burrowing and entrapment.*

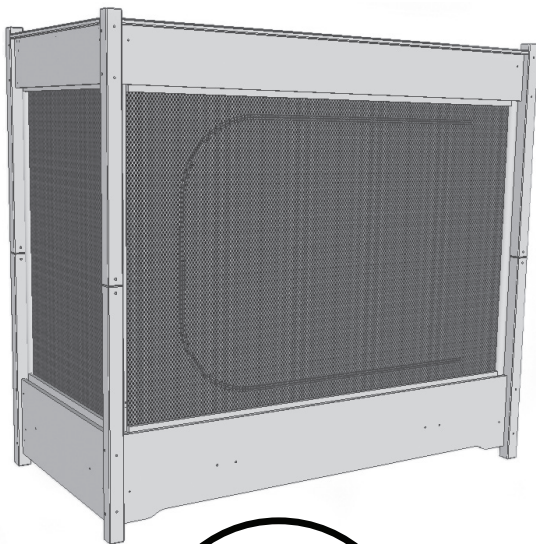

**SleepSafeBed.com | 866.852.2337**

3629 Reed Creek Drive | Bassett, VA 24055

# Table of Contents

- 1 Pre-Assembly Information
- 2 Component Chart
- 3 Assembly Guide: Bottom Side Panels
- 4 Assembly Guide: Top Side Panels, Corner Brackets
- 5 Assembly Guide: Top Side Panels, Preparing SuperScreen Surround
- 6 Installing SuperScreen Surround
- 7 Installing SuperScreen Surround, Aluminum Cleats, Cross Supports
- 8 Installing Mattress/Bunkie Combination
- 9 Installing SuperScreen Top
- 10 Mattress Specifications
- 11 General Information
- 12 Warranty Information
- 13 DME Provider / Technical Support, Serial Number



## Need Help?

Scan this QR Code to view step-by-step video assembly on our YouTube channel.

*This guide is available as a PDF. Contact Customer Service at 866.852.2337 for more information.*

## Tools Needed for Assembly



Allen Wrenches  
(Supplied)



Phillips and Flat Head  
Screwdrivers



3/8" Open or Closed  
End Wrench



Two People for Assembly

# Attention! Read Before Assembly

- **KEEP THIS PRODUCT GUIDE.**

IT CONTAINS ASSEMBLY INFORMATION  
AND THE BED'S SERIAL NUMBER

-----

- CAREFULLY REMOVE AND LAY OUT ALL  
COMPONENTS. FAMILIARIZE YOURSELF  
WITH INSTRUCTIONS TO FACILITATE  
THE ASSEMBLY OF YOUR BED.
- 

- CHECK COMPONENTS AND  
CONTENTS OF ENCLOSED HARDWARE  
BAG WITH THE HARDWARE LIST ON  
THE FOLLOWING PAGES.
- 

- DUE TO THE PRESENCE OF SMALL  
PARTS DURING ASSEMBLY, KEEP OUT  
OF REACH OF CHILDREN.
- 

- FOLLOW THE INSTRUCTIONS IN THE  
SPECIFIED ORDER TO ENSURE THE  
BED IS PROPERLY ASSEMBLED.
- 

- COMPLETE ASSEMBLY. THE SUPER-  
SCREEN TOP MUST BE INSTALLED  
FOR PROPER INSTALLATION AND  
STABILITY. DO NOT OMIT ANY COM-  
PONENT OR PROCESS.

- NEVER SUBSTITUTE OR ALTER PARTS,  
ONLY COMPONENTS AND HARDWARE  
SHIPPED WITH THE BED ARE USED  
FOR ASSEMBLY.
- 

- CHECK THAT PADDING IS SEWN AND  
FIRMLY ATTACHED TO THE SUPER-  
SCREEN SURROUND AND TOP.
- 

- CAUTION: POTENTIAL BED TIPPING:  
THE INSIGHT BED HAS AN  
ANTI-TIPPING STRAP ATTACHED TO  
THE HEADBOARD. IT IS RECOMMEND-  
ED TO ATTACH THIS SECURELY TO THE  
WALL FOR STABILITY.
- 

- DO NOT SUBSTITUTE MATTRESS.  
MATTRESS IS SPECIFICALLY DESIGNED  
WITH THE BUNKIE BOARD AND FITTED  
FOR AN INSIGHT™ BED
- 











- NEVER USE THIS BED IF THERE ARE ANY  
MISSING FASTENERS, LOOSE JOINTS,  
BROKEN PARTS, TORN MESH, OR  
PADDING INADEQUACIES. CONTACT  
SLEEP SAFE BEDS AT 866.852.2337 FOR  
ASSISTANCE.



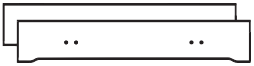






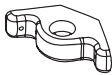

**SleepSafeBed.com | 866.852.2337**

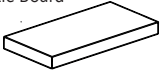3629 Reed Creek Drive | Bassett, VA 24055

# Included Components for Assembly

|   |    |
|---|----|
| Aluminum Side Cleats<br>             | 2  |
| Aluminum End Cleats<br>              | 2  |
| 1" by 1" Aluminum Cross Supports<br> | 2  |
| 25 mm Socket Bolts<br>               | 4  |
| 3/4" Frame Mounting Bolts<br>        | 12 |
| 90 mm Allen Bolts<br>                | 16 |
| .875 Cross Slot Trimmer Screws<br>   | 16 |
| Barrel Nuts<br>                    | 16 |
| Corner Support Brackets<br>        | 8  |
| 4mm Allen Wrench<br>               | 1  |

|  |   |
|--|---|
| Mattress/Bunkie Board Combination<br>                     | 1 |
| Headboard and Footboard (Set)<br>                         | 1 |
| Bottom Side Panels<br>                                    | 2 |
| Top Side Panels<br>                                       | 2 |
| SuperScreen Top<br>                                       | 1 |
| SuperScreen Surround (2 parts to be zipped together)<br> | 1 |
| Padded Corner Supports<br>                              | 4 |
| Casters (Optional Installation)<br>                     | 4 |
| Anti-Tip Kit Hardware (Partially installed)<br>         | 1 |



FIXED FOUNDATION



**SleepSafeBed.com**  
**1-866-852-2337**

3629 Reed Creek Dr. | Bassett, VA 24055

# InSIGHT™ Bed Assembly Guide



*After unpacking bed and checking the contents.*

## 1 Insert Barrel Bolts into Top and Bottom Side Panels.

Align and push in Barrel Nuts as shown. Use flathead screwdriver for assistance.

**Each panel end needs 2 Barrel Bolts inserted.**

*Align hole in Barrel Bolt to hole in Side Panel*

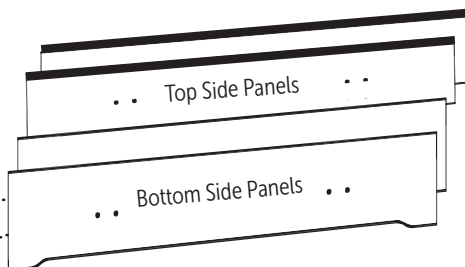
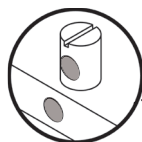


*A total of 16 Barrell Bolts need to be inserted.*

## 2

Footboard


Headboard

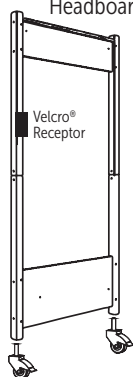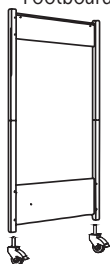

## Position Headboard and Footboard to prepare for Bottom Side Rail installation.

Two installers are recommended for this stage of assembly. Be sure that the headboard and footboard are aligned properly.

*Before installation, be sure that the rubber sleeve on the stem is touching the top metal nut.*

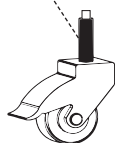


### Caster installation (Optional)

Insert the stem into the hole in the bottom of the leg, first lock the caster, and then turn the entire caster clockwise until it expands and is securely in place..

## Attach Bottom Side Panels.

With 4mm Allen Wrench, use (8) 90mm Allen Bolts to fasten both bottom side panels to the footboard and headboard. Leave bolts slightly loose, as they will be tightened later.

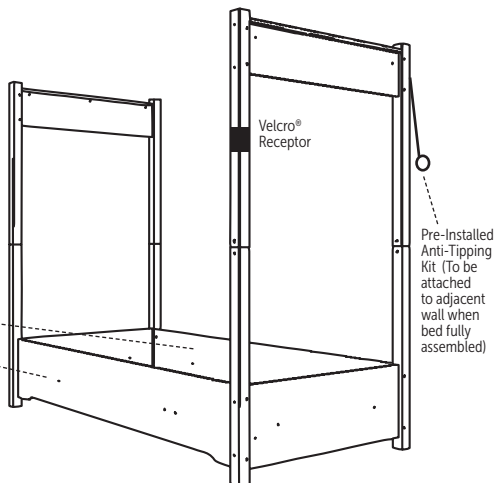


# InSIGHT™ Bed Assembly Guide

3

## Attach Top Side Panels.

With 4mm Allen Wrench, use (8) 90mm Allen Bolts to fasten both top side panels to the footboard and headboard.

Leave bolts slightly loose, as they will be tightened securely later.

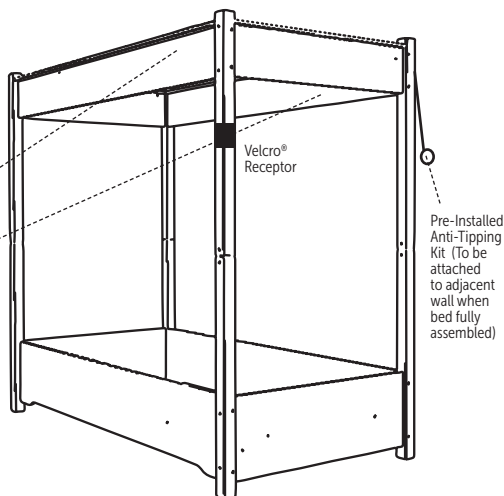


4

## Attach Bottom Corner Brackets.

Use phillips head screwdriver with .875 cross slot trimmer screws to fasten Corner Brackets into each corner. Use two screws for each bracket.

## Insert end of Padded Corner Supports into Bottom Brackets

Insert the end of the Padded Corner Supports into the Bottom Brackets. Install one padded corner support at a time. Then, see step #5.

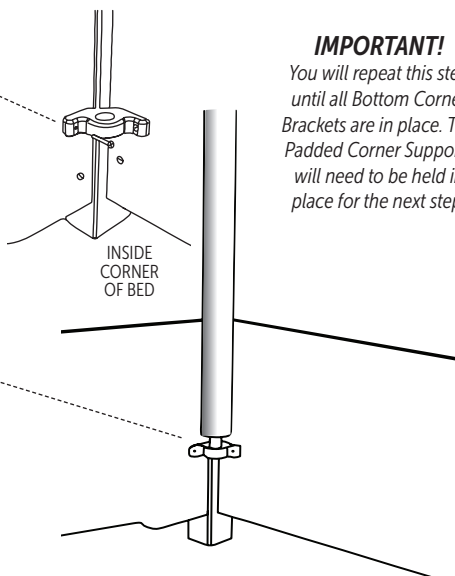

### IMPORTANT!

You will repeat this step until all Bottom Corner Brackets are in place. The Padded Corner Supports will need to be held in place for the next step.



FIXED FOUNDATION




SleepSafeBed.com

1-866-852-2337

3629 Reed Creek Dr. | Bassett, VA 24055

5

Insert end of  
Padded Corner  
Support into  
Top Corner  
Bracket.  
Fasten Bracket  
to the bed.

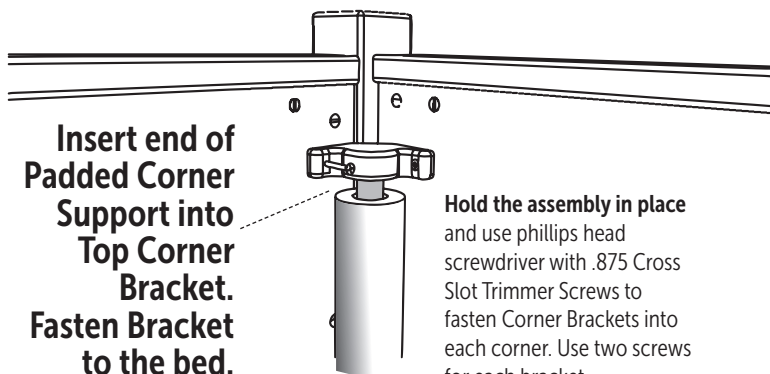

Hold the assembly in place  
and use phillips head  
screwdriver with .875 Cross  
Slot Trimmer Screws to  
fasten Corner Brackets into  
each corner. Use two screws  
for each bracket.

**IMPORTANT!**

*You will repeat this step until  
all four Padded Corner Supports are installed.*

6

Zip together both  
parts of SuperScreen  
Surround to prepare  
for installation.

**NOTE:** AA & BB mark the  
points where the parts match  
when zipped together.


Take care to insure proper  
configuration of the  
Surround in this step.

*Two installers are recommended  
to handle the surround as it is  
heavy and awkward to handle.*

After you zip  
together the  
parts, LOCK  
ALL 4 BOTTOM  
CORNER TABS  
WITH THE  
S-BINER.



TOP: Trim-Lok snaps to bed caps

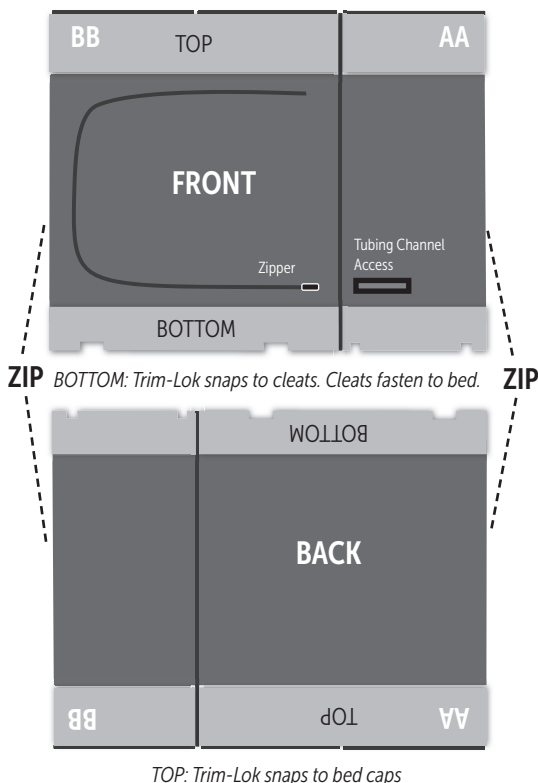

# InSIGHT™ Bed Assembly Guide

7


Place the SuperScreen Surround into the bed frame and lift it to the top.

Find the TRIM-LOK stitched into the top edge of the Surround.

First, SNAP INTO THE GROOVES in the caps of the Headboard and Footboard.

Then, starting in the center, snap the Trim-Lok into the grooves in the Side Panel caps.

All TRIM-LOK should be tightly snapped in place.

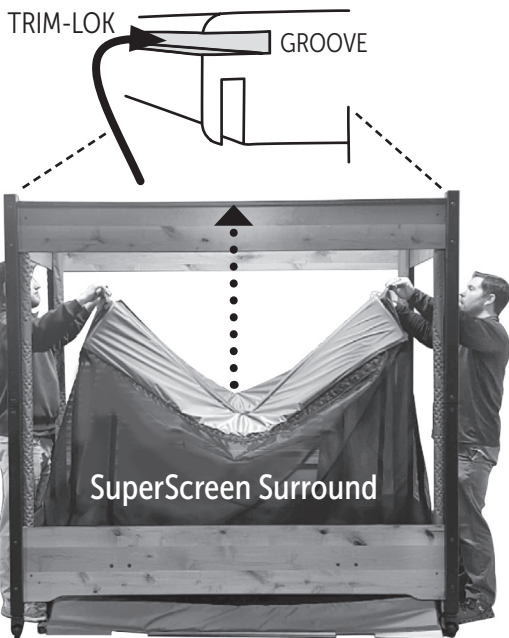

*TRIM-LOK snaps the Surround completely around the bed frame.*





8

Unzip the screen to access the inside of the bed.

Snap TRIM-LOK from Surround to bottom edge of Aluminum Cleats

Note: 4 Cleats and 2 Cross Supports need to be installed from inside the bed frame.



### IMPORTANT! FOLLOW STEPS A,B,C

Securely SNAP the TRIM-LOK to the bottom edge of the Aluminum Cleats before fastening the cleats to the bed panels.

A

TRIM-LOK SNAPS TO BOTTOM EDGE OF CLEAT.



B


FIRST, USE 3/4" BOLTS TO FASTEN END CLEATS TO HEADBOARD AND FOOTBOARD PANELS. THEN, FASTEN SIDE CLEATS TO SIDE PANELS.



*Fasten the inside bolts loosely then fasten the outside bolts. Now tighten them all.*

C

LASTLY, USE 25MM SOCKET BOLTS TO FASTEN THE TWO CROSS SUPPORTS TO THE SIDE ALUMINUM CLEATS.

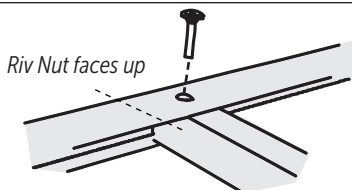

FIXED FOUNDATION

**InSIGHT**<sup>TM</sup>  
by SleepSafe® Beds

**SleepSafeBed.com**


**1-866-852-2337**

3629 Reed Creek Dr. | Bassett, VA 24055

# InSIGHT™ Bed Assembly Guide

9

**Install the  
Mattress/Bunkie Board  
Combination.**

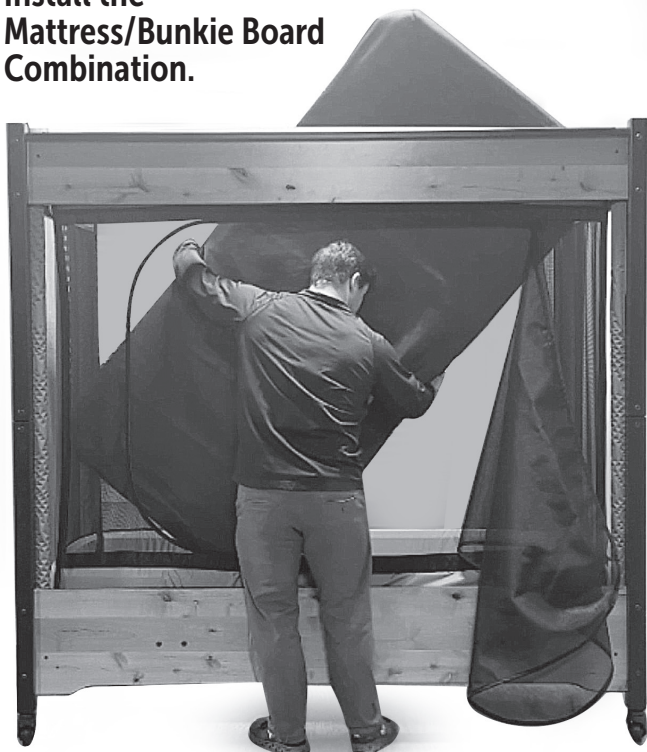

**The Mattress/Bunkie Board combination is not flexible and is more challenging to install than a normal flexible Mattress.**

As pictured, it is best to lift one end into the bed at a time, then twist it down to rest firmly on the aluminum cleat foundation.



FIXED FOUNDATION



**SleepSafeBed.com**

**1-866-852-2337**

3629 Reed Creek Dr. | Bassett, VA 24055

**DO NOT SKIP  
THIS STEP!**

**10** Place the  
top onto the  
bed frame.



Find the  
**TRIM-LOK**  
stitched into  
the edges.

Starting with  
the Headboard  
and Footboard,  
**SNAP TRIM-LOK**  
into the bottom  
side of the caps.

Then attach to  
the side panels,  
starting in the  
middle, working  
outward.

**ANTI-TIP HARDWARE**

Install necessary hardware  
for pre-installed cord  
to be attached to the wall  
to complete assembly.

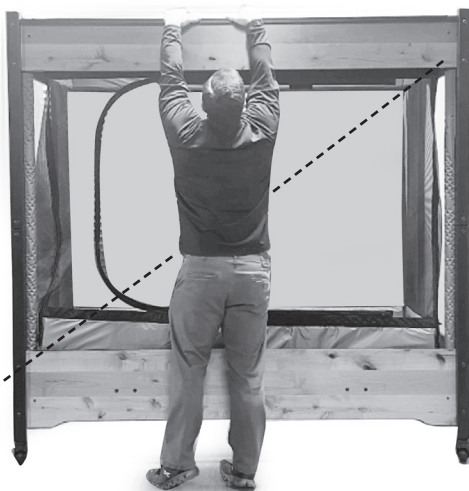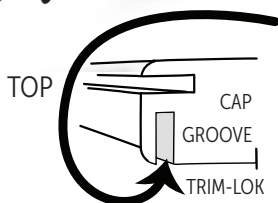

***Double Check: TIGHTEN ALL HARDWARE!***

# InSIGHT™ Bed Mattress Information


## SleepSafe Bed® – Our Memory Foam Mattress with Gel Technology

SleepSafe® Bed mattress support surface provides optimized pressure redistribution, rapid heat dissipation, and increases breathability compared to standard foam.

- Reduced incidence of pressure ulcers by channeling heat away from the body much faster than standard foam, virtually ridding perspiration moisture.
- Superior pressure redistribution achieved by providing more support and less cradling at higher levels of compression, distributing pressure over a larger surface area and eliminating static fatigue.

### How Does It Work?

The technology works by allowing air to pass through open cells, channeling it away from the body instead of trapping it within the closed cells of traditional memory foam. The infusion of gel polymers throughout the foam contributes both cooling properties and the virtual elimination of static fatigue. As a result of the marriage between open cell foam and gel infusion, the mattress holds its shape better, redistributes pressure over a larger surface area, and successfully wicks heat away from the body, leaving patients cool, dry, and comfortable.

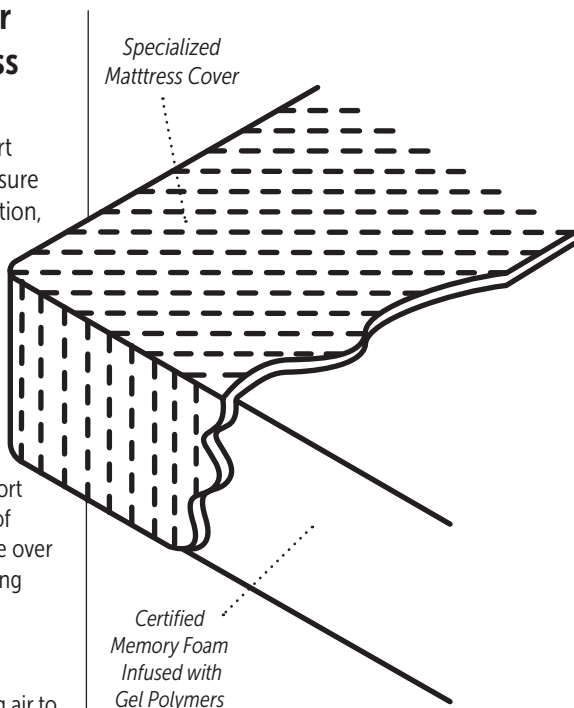

### Foam products meet the following criteria:

- |                         |                                      |
|-------------------------|--------------------------------------|
| • Low Emissions (VOC's) | • Performance – Tested               |
| • BPA Phthalate – Free  | • Made without Ozone Depleters       |
| • CFC – Free            | • Mercury, Lead and Heavy Metal Free |
| • BHT – Free            | • Made without Formaldehyde          |
| • PBDE – Free           | • Methylene Chloride – Free          |
| • Emissions – Tested    | • Environmentally Innovative         |
| • Content – Tested      | • Material Certification             |

### Other notes about the foam in mattresses:

- Foam products use a blend of natural oil-based and conventional polyols.
- Benefits farmers looking for a new market for crops.
- Polyol is made partially from plant-based renewable oils such as soy bean and castor beans.



FIXED FOUNDATION




**SleepSafeBed.com**

**1-866-852-2337**

3629 Reed Creek Dr. | Bassett, VA 24055

# InSIGHT™ Bed General Information

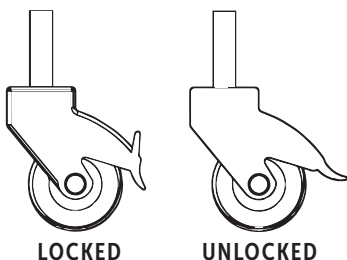

## How Casters Work.

The casters are not installed at the factory but are available to insert into the bottom of the headboard and footboard legs. To install casters, lock and turn the entire caster clockwise until secure. The casters should be locked with the tab down when the bed is in use to add to the bed's stability. The tab is in the down position and unlocked when the tab is up.

## About SuperScreen

SuperScreen passes as lead free, phthalate free and meets "Greenguard Gold" safety standards, ensuring the products are free from harmful volatile organic compounds (VOC's) and safe for your family. Super Screen is extremely tough! It also protects against the sun's harmful ultraviolet rays.

For cleaning, a little dish detergent mixed with water works wonderfully using a sponge. Avoid abrasives, bleach or heavy chemicals.

## Padding Information

### Materials:

- 70 Denier Nylon Fabric Cover with a Fluid Resistant Healthcare Coating
- 3/4" High Density Foam Padding

### Padding Features:

The padding on the InSIGHT™ Bed is part of the SuperScreen surround. It is sewn into the bottom and top of the surround and is fastened with TRIM-LOK to the top and bottom of the bed frame.

- Padding covers the corner supports.
- Anti-Static
- Non-allergenic
- Bacteria Resistant
- Stain Resistant
- Malodor Protection
- Fluid Resistant
- Fungal Resistant
- Flame Retardant
- Tear Resistant

## Padding Maintenance

- For cleaning, use warm water and a mild cleanser. Wipe surface gently.

## Important Note:

- Padding is installed at the factory and will cover ALL 4 sides. Any alteration to the padding will void the warranty.

## Padding questions?

- Please contact customer support for any specific information about padding.

# Bed/Mattress Warranty Information

## **Full One-Year InSIGHT™ Bed Warranty**

The InSIGHT™ Bed is warranted against defects in workmanship and components for a period of one year from the date of purchase. SleepSafe® Beds, LLC will repair or replace any defective part at no cost to the purchaser if the shipping of the part is within the continental United States.

## **Limited Five Year InSIGHT™ Bed Warranty**

During the second through fifth years from the date of purchase, SleepSafe® Beds, LLC will replace any part found to be defective. Purchaser shall pay all service, repair, and shipping costs related to the replacement of the defective part.

## **Terms and Conditions for InSIGHT™ Bed Warranty**

An InSIGHT™ Bed is expressly prescribed for a single user. This warranty will cover the bed for this user only. Warranty coverage beyond this use is handled on a case by case basis. This warranty applies to normal use and does not cover any damage caused by excessive wear, abuse, misuse, mishandling or modification of the product.

## **InSIGHT™ Bed Mattress Warranty Information**

All InSIGHT™ Bed models are shipped with a CertiPUR-US® memory foam mattress with gel infused technology. This surface supports up to 350 lbs. and provides optimized pressure redistribution, rapid heat dissipation and increased breathability. Memory foam infused with gel polymers is designed for patient comfort as well as clinical superiority. Benefits include: reduced incidence of pressure ulcers by channeling heat away from the body faster than standard foam, virtually eliminating perspiration moisture. The mattresses are water resistant, but are not waterproof.

## **Full One-Year Warranty for InSIGHT™ Bed Mattress**

Full one year warranty from date of purchase. SleepSafe® Beds, LLC will replace the mattress due to a defect in the materials and workmanship of the mattress, at no cost to the purchaser.

## **Limited Five Year Warranty for InSIGHT™ Bed Mattress**

During the second to fifth year, SleepSafe® Beds, LLC will replace the mattress due to a defect in the materials or workmanship of the mattress, with the shipping costs to be paid by purchaser.

## **Terms and Conditions for InSIGHT™ Bed Mattress Warranty**

Any breakdowns or deterioration of the product caused by factors other than a defect in the materials and workmanship of the mattress, such as repeated exposures to bodily fluids by incontinence, are not covered by this warranty.

## Durable Medical Equipment Provider

Bed Serial # \_\_\_\_\_

Installation Date \_\_\_\_\_

DME Provider \_\_\_\_\_

DME Contact \_\_\_\_\_

DME Address \_\_\_\_\_

DME Phone \_\_\_\_\_

DME Email \_\_\_\_\_

Installed by \_\_\_\_\_

## Customer Service & Technical Support

### SleepSafe Bed Support

Hours: 8 AM to 5 PM / M-F / EST

Toll Free: 866.852.2337

Local: 276.627.0088

Fax: 276.627.0234

Web: SleepSafeBed.com

*In case of an EMERGENCY after hours, please call 866-852-2337 for instructions on how to access the on-call technician.*

### DME Provider Support

DME: \_\_\_\_\_

Contact: \_\_\_\_\_

Phone: \_\_\_\_\_

Email: \_\_\_\_\_

Web: \_\_\_\_\_



**SleepSafeBed.com | 866.852.2337**

3629 Reed Creek Drive | Bassett, VA 24055

WI-820.120-23 Rev. 03